

JBP V LIFT

Radically New Approach for Anti-Aging Treatment

What is “JBP V Lift” ?

PDO
(Polydioxanone)



- ◆ Embedding therapy needle with absorbable suture (PDO)
- ◆ Injecting several dozen of needles on cheeks one by one.
- ◆ After pulling needles out, the inserted suture stay into the skin.
- ◆ Stimulate the hypodermal tissue and increase collagen synthesis



- Show Quick Face Lifting Effect without “Surgical” treatment
- No “Down Time” for injection with very little pain, swelling and injection mark
- Continuing Effect: 6 - 8 months
(Suture dissolves naturally and harmlessly in the skin)
- Officially approved by KFDA, CE Marking and US FDA(Spring of 2013)

The product lineup of JBP V LIFT

Needle Gauge	Length of Needle	Length of Thread	Thread USP Size
25G	90 mm	150mm	5-0
27G	60 mm	90mm	5-0
29G	50 mm	70mm	6-0
29G	40mm	50mm	6-0
31G	30mm	30mm	7-0

Enable to Use different variety depending on patient's condition and request

The process of JBP V Lift Treatment

1. Counseling

Consult about injection sites and number of needles depending on patient's request and condition.
For a single treatment, inject about 25 - 50 needles for one cheek with crossing or upward.



2. Remove makeup and put marks

3. Apply anesthetic cream

Wait for 30-60 min. for anesthesia to work.

*In some patients, the use of nerve block injection is recommended.
(By JBP Nanoneedle 34G)



The process of JBP V Lift Treatment

4. Before inserting the needles, soak PDO suture in **Laennec** (JBP Human Placental Extract) to promote “Wound Healing”, “Fibroblast Activation” and “Cell Rejuvenation”.



Use 2 to 3 ampoules for one treatment

The process of JBP V Lift Treatment

5. After remove anesthesia cream, inject V Lift upward into the skin quickly. The treatment of the one side (25 needles) can be finished within 3 minutes.



6. Soon after injecting all the V Lift for the one side, pull out the needles quickly. (The inserted suture stay into the skin.)



The process of JBP V-Lift Treatment

7. To avoid the internal bleeding and swelling, **compress the injected area with sterile gauze forcefully** (About 2 to 3 min.)



**Very little pain, swelling and injection mark.
Patients can make up just after the treatment.**

The process of JBP V-Lift Treatment



JBP V Lift is also effective for sagging neck.

Insert 15-20 needles depending on patient's condition with avoiding artery.

Merits of JBP V Lift Treatment

● Fast-Acting

Quick effects for face-lifting right after treatment

● Safety

Suture dissolves naturally and harmlessly in the skin within 6 months

● Stress-Relief

Embedding needle therapy without surgery downtime.

● Quality

Officially approved by KFDA, CE Marking and US FDA(Spring of 2013)